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Battery powered height adjustable trolley. Desktop light grey HPL laminate with dark grey ABS edges. Steel frame, 2 pcs fully welded parts with engine between. 125 mm wheels, 2 pcs with breaks. Li-Ion battery 1800 mAh. Including power supply. Height adjustment with fixed control unit.

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<i>Type:</i>	<i>Adjustable trolley</i>	<i>Height adjustm.:</i>	<i>700 - 1000 mm</i>
<i>Art no:</i>	<i>2010410</i>	<i>Maximum load:</i>	<i>150 kg</i>
<i>Desktop size:</i>	<i>800x600x24 mm</i>	<i>Wheel:</i>	<i>125 mm (2 pcs breaks)</i>
<i>Desktop:</i>	<i>HPL laminate</i>	<i>Engine:</i>	<i>Phoenix Mecano</i>
<i>Frame:</i>	<i>Steel profiles</i>	<i>Battery:</i>	<i>Li-Ion 1800 mAh</i>
<i>Frame Surface:</i>	<i>Powder coating</i>		

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The Trolley's height adjusts with the control unit. The UPP or DOWN button shall be kept down the entire time you want to adjust the height. As soon as the button is released the height adjustment stops. The Trolley should never be adjustable in height during movement. The wheels should be locked when height adjustment.

During transport the Trolley should be in the lowest point for extra stability.

Specified maximum weight is specified for evenly distributed load. Never load the table unevenly to avoid the risk of tipping.

Always load the battery with the following charger. See specific document for more information regarding the battery.

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The Trolley delivers fully mounted and doesn't requires any mounting after delivery. Control that the table is functional after delivery. If needed charge the battery before use.

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The manufacturer provides 2 years guarantees on electric system such as motor and control unit. Wheels and desktop have 2 years guarantee under normal use. The welded frame has 5 years guarantee.

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Vi, Nyblads i Västerås AB, försäkrar härmed under eget ansvar att produkten:
We, Nyblads i Västerås AB, hereby declare during our own responsibility that the product:
VD Nyblads AB